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HOW GRADUATED COMPRESSION WORKS

In-depth studies in the field of haemodynamics have shown that stockings with decreasing graduated compression help to ensure a proper return blood flow towards the heart. This micro-massage effect helps to prevent water retention and varicose veins, while at the same time improving muscle oxygenation.

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

Pre-shaped socks, gradually compressed from the ankle to the calf, are recommended during and after sports activity, for optimal recovery of venous return into the bloodstream.

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Travel Relax
Walking Job
Meeting
Casual

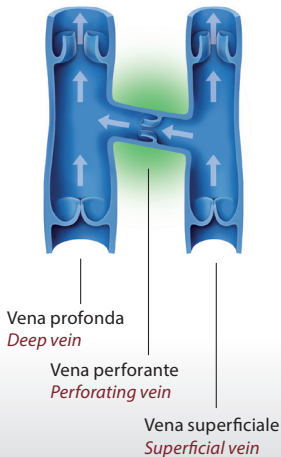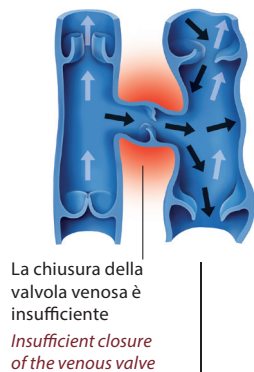**CIRCOLO VENOSO SANO
HEALTHY VEIN CIRCULATION****VENE VARICOSE
VARICOSE VEINS**

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GRADUATED COMPRESSION, THE REAL ONE

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EN

LET'S TAKE CARE OF OUR SUPPORT POINT

Dear Customer, thank you for having chosen this Human Compression Wear product. Our products are synonymous with elegance, comfort and, above all, personal wellness.

The pleasant feeling of lightness and wellness is obtained thanks to the dynamic effect of the product's graduated compression. The circulation is gently stimulated in order to facilitate venous return. The compression decreases from the ankle upwards.

The heel section provides for:

- the proper placement of the heel in the heel pocket, thus providing for greater stability.
- proper positioning at the maximum compression point, as well as a better fit and greater wearability and elasticity around the circumference of the ankle.

Directions

Human products are useful for preventing venous disorders. They are particularly effective at preventing fatigue and a sense of heaviness in the legs. They are designed to provide a pleasant sensation of comfort throughout every daily activity.

General contraindications

Advanced state peripheral arterial occlusive disease - Massive oedema of the lower limbs of a pathological nature - Septic phlebitis - Severe phlebitis, such as phlegmasia cerulea dolens - Lymphangitis - Advanced peripheral neuropathies (i.e., diabetes mellitus) - Other sensory disorders of the limbs - Intolerances/allergies related to the material of the compressive stockings.

Relative contraindications

Ulcers and open wounds - Moist dermatosis - Immobility - Dermatitis stasis - Valvular incontinence of the superficial and deep venous system. - Deep vein thrombosis or acute

thrombophlebitis - Post thrombophlebitic syndrome - For stockings with silicone edges, skin irritation may arise in cases of hypersensitivity to the material itself.

Composition of the materials

The textile composition of each individual product is indicated on the outside of the package. The materials with which our products are manufactured are listed hereafter.

CO: Cotton - **EA:** Elastane - **PA:** Polyamide

WO: Wool - **PP:** Polypropylene - **SE:** Silk

Washing instructions

Human stockings can be washed by hand or in the washing machine.

- Hand washing: use detergents for delicate fabrics. Rinse thoroughly. Do not wring. Air dry.
- Washing machine: place the garment in an appropriate bag or in a pillow case. Do not bleach. Do not machine dry.
- Do not dry clean. Do not iron.


Warning

The manufacturer shall bear no responsibility for any personal injuries and/or property damage that may occur in the event that the product is used improperly or for purposes other than those for which it is intended.

Instructions and recommendations for wearing Human sock

Protect the socks against contact with any oils, creams, and ointments that may be applied to the legs. Rough fingernails or rings with rough edges could potentially damage the product's fabric. If the skin is wet, apply talcum powder to the foot in order to facilitate its insertion into the sock.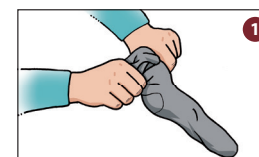

1. Turn the sock inside-out up to the heel - Use both hands to widen the foot portion and slide the foot into the sock up to the heel.

2. Turn back the inside-out portion of the sock, taking care to position the heel properly.

3. Pull the rest of the sock up to the knee, smoothing out the fabric with the palm of your hand. Never pull directly on the top edge of the sock.